Hainuo Group has been repeatedly awarded as one of the "Top Three Enterprises in China's Band-Aid Industry", and cartoon Band-Aids pioneered the domestic band-aid industry. The company's main products are: face masks, wound stickers, medical infusion stickers, pressure-sensitive adhesive tape, medical disinfection cotton balls, medical disinfection cotton pads, trauma emergency kits, syringes, infusion sets, early pregnancy test paper, ovulation test paper, foot stickers, warm patch, medical antipyretic stickers, gauze bandages, disinfectant, dental floss sticks, hyaluronic acid toothpaste, medical care pads, ice pads, condoms, etc.

HYNAUT

HAISHI HAINUO GROUP

Qingdao Hainuo Biological Engineering Co., Ltd.

Qingdao HYNAUT IMP. & EXP. Co., Ltd.

About us

Hainuo Group was founded in 2004 and now has more than 20 subsidiary corporations. Our headquarter is located in Jiangshan industrial park in Laixi, Qingdao.

We have a dozen up-to-date factories, multiple Class 100,000 cleanrooms, modern logistics / storage facilities, advanced production facilities and detecting instruments. More than 2,000 professional staffs devote to the development of Hainuo, including experts, professors and licensed pharmacists from Chinese Pharmaceutical Association and the State Council, enjoying the technology support and cooperation of many research institutions / companies from USA, Germany and Taiwan, China.

Hainuo keeps moving forward and we're always open to your ideas and advices.

FILTERING HALF MASK

FFP2

Product name: Filtering Half Mask

Model: LX1001

Primary structure: 5 layers

Validity: 3 Years

Application:

It is used in the dust protection industry of construction, metal casting, stone ore, electronics, pharmacy, physical treatment and grinding, and has good protection effect on dust storm, haze and PM2.5.

Storage:

Normal atmospheric temp. < 40°C.

Maximum relative humidity under 80%

FFP2 WHITE LIST : CE, EUA

PACKING DETAILS

PKG	5Pcs/Bag,10Bags/Box,16Boxes/Ctn
N.W.	AR.8KG
G.W.	AR.9KG
SIZE	64*40*38CM

Warnings

Before use the masks first verify that it is suitable for the intended use. The wearer must be adequately trained prior to use and ensuring the masks are proper fit.

The mask is disposable and cannot be reused.

Using the masks with facial hair will cause leakage problem, if the facial hair not covered, unlikely to achieve seal.

Discard and replace the mask if it becomes damaged or higher breathing resistance.

Don't use the mask in explosive gas, heavily polluted environment, oxygen-deficient environment, fire environment and the underwater work.

Patients with heart disease or other disease should discard it after wearing uncomfortably.

Mask marked 'NR' cannot be used for > one shift.

This product doesn't supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19.5%.

Do not use in explosive atmospheres.

It's strictly prohibited to use if the package is damaged.

Due to personal differences, glasses may be foggy, so please pay more attention when driving.

As this product is a disposable mask product, it cannot be reused by cleaning.

Keep out of reach of children.

Please pay attention to avoid high temperature and humidity during storage, and keep in a clean place.

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OEM, ODM

Step 1:

Open the mask, with the bridge of the nose facing up.

Step 2:

Hang the face mask with its rope on the ears, and make sure your nose and chin are covered with it as well as the face.

Step 3:

Press the part of the face mask tightly which is close to the nose to make sure it is fully covered well.

Step 4:

Press the face mask on the face tightly, and make sure it is very close to the contour of your face.

HYNAUT

Excellent products of Hainuo create a healthy and beautiful life.

CONTACT

TEL: 0086 532 86999302

CEL: 0086 15065420708

MAIL: INFO@HYNAUT.CN

WEB: WWW.HAINUOCN.COM WWW.HYNAUTGROUP.CN

ADD: NO. 1, GUANGDONG RD, JIANGSHAN INDUSTRY
PARK, LAIXI, QINGDAO, CHINA

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Hersteller: Haishi hainuo lexiang medical technology (Qingdao) Co.,Ltd