

青岛海氏海诺集团

产品册——人绒毛膜促性腺激素（HCG）测定试纸

Product catalogue—Human Chorionic Gonadotropin(HCG) Test Strip

QINGDAO HAINUO GROUP

About us

Hainuo Group was founded in 2004 and now has more than 20 subsidiary corporations. Our headquarter is located in Jiangshan industrial park in Laixi, Qingdao, which is very close to the main highway, Huashan International Golf Course, Qingdao International Airport and Qingdao Harbor. Our factory covers an area of 220,000 square meters with super geographical position and convenient transportation.

We have a dozen up-to-date factories, multiple Class 100,000 cleanrooms and Class 300,000 cleanrooms, modern logistics and storage facilities, advanced production facilities and detecting instruments. More than 2,000 professional staffs devote themselves to their job and the development of Hainuo, a lot of them are experts, professors and licensed pharmacists from Chinese Pharmaceutical Association as well as the State Council. We also have technology support and cooperation of many research institutions and companies from USA, Germany and Taiwan, China.

Hainuo keeps moving forward and we're always open to your ideas and advices. Please don't hesitate to let us know if you're interested in working with us and a deeper level of cooperation!

OUR BRANDS



HYNAUT for Overseas Market



HAINUO for Domestic Market

HYNAUT HCG试纸采用胶体金法，是一种简单可靠的早期妊娠测试方法。它不需要任何特殊的设备和试剂，并且可以在很短的时间内提供直观而精确的诊断。

HYNAUT HCG Test Strip uses colloidal gold method, which is a simple and reliable method for testing early pregnancy. It doesn't require any special equipment and reagent, and it can provide an intuitional and precise diagnosis in a very short period of time.



- **使用说明**

将试纸浸入尿液10秒钟。不要浸入最大线。

将带子放平并等待。

在3-10分钟内读取结果。

Description of Use :

Immerse the strip into the urine for 10 seconds. Do not immerse past the Max line.

Lay the strip flat and wait.

Read result in 3-10 minutes.

- **如何判定结果**

阳性结果：将出现两条粉红色的线（线条强度可能会有所不同）。

阴性结果：C处将出现一条粉红色的线。

无效的结果：没有可见的线条出现。重复进行新的测试。

Reading Results:

Positive results: Two pink lines will appear. (intensity of lines may vary)

Negative results: One pink line will appear at C.

Invalid results: No visible lines appear. Repeat with a new test.

产品图

Product Photo

条型
Strip Type



Model: C351

Size: 2.5mm

卡型
Cassette Type



Model: C352

Size: 3mm

笔型
Pen Type




Model: C353

Size: 6mm

包装效果图 条型


Packing Effect Drawing Stripe Type

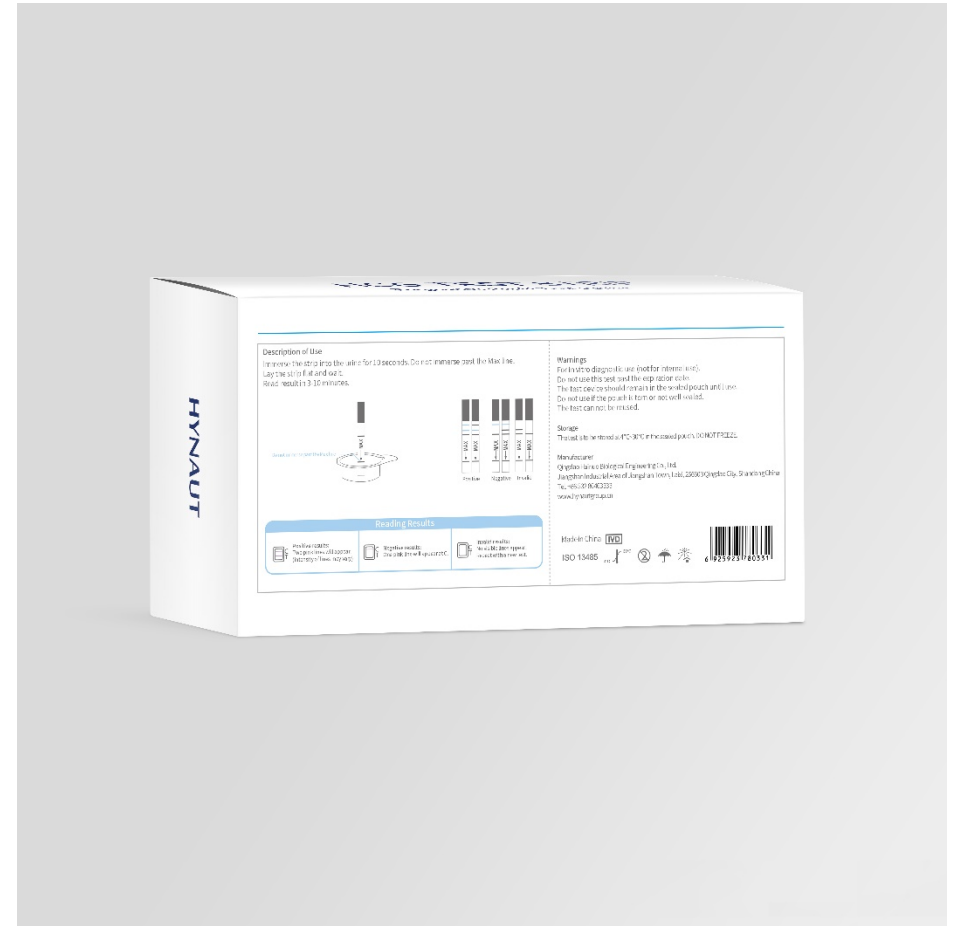

100pcs/box

包装效果图 卡型

Packing Effect Drawing Cassette Type

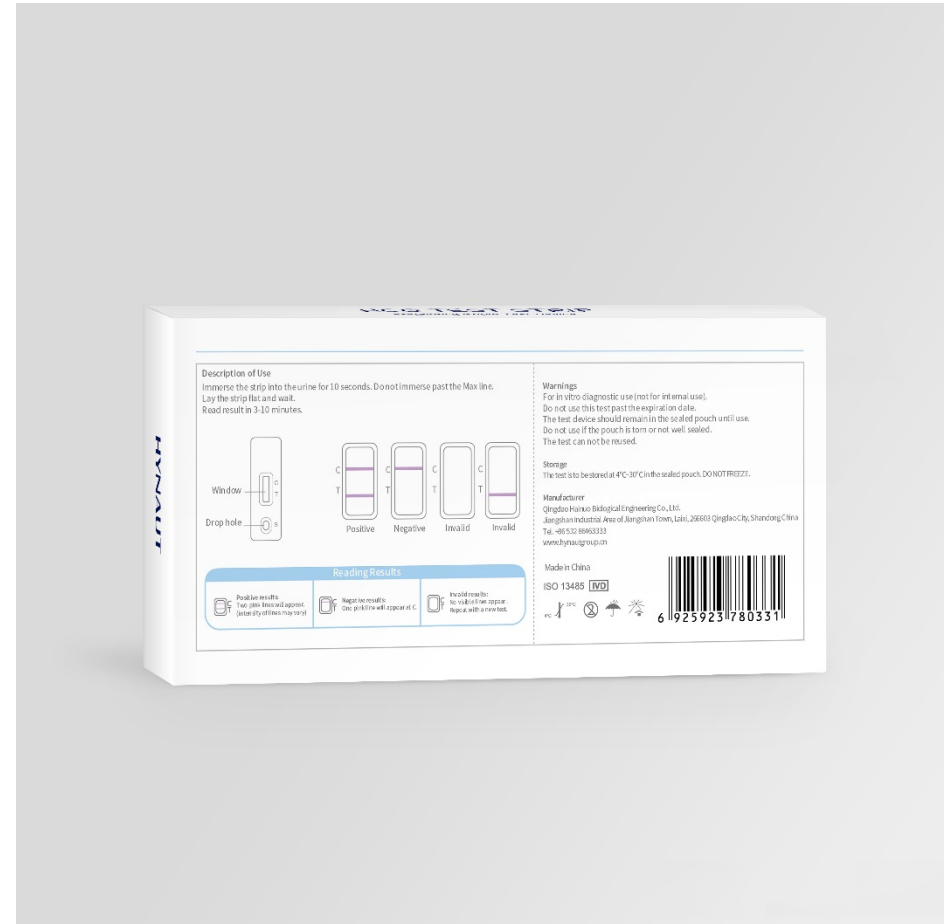

1 pc/box

包装效果图 笔型

Packing Effect Drawing Pen Type



1 pc/box

箱规

Packing Size

	铝箔袋袋尺寸 Foil Bag Size	盒子尺寸 Box Size	外箱尺寸 Carton Size	规格 Packing Size
条型 Strip Type	11.5*5.5cm	23*12.3*6cm	64.5×48.5×32.0cm	100pcs/box, 50boxes/carton
卡型 Casette Type	12*6cm	14*7*2cm	59.0×42.5×45.5cm	1pc/box, 480boxes/carton
笔型 Pen Type	18.5*6cm	19.5*7*2cm	61.0×44.0×58.5cm	1pc/box, 480boxes/carton

相关证书 ISO 13485

Certification ISO 13485



2020-09-03

To Whom It May Concern

This is to confirm that Extension Audit, Re-certification Audit for ISO 13485, Surveillance Audit for MDD was carried out on behalf of TÜV Rheinland LGA Products GmbH Notified Body (CE0197) as follows:

Applicant: Qingdao Hainuo Biological Engineering Co., Ltd.
Address: Jiangshan Industrial Area of Jiangshan Town, Laixi, 266603 Qingdao City, Shandong, China

MDDScope: Aspects of manufacture concerned with securing and maintaining sterile conditions of Sterile Wound Plasters, Sterile Wound Dressings, Medical Face Masks

qsScope: Manufacture and Distribution of Sterile Wound Plasters, Sterile Wound Dressings, Medical Adhesive Tapes, Surgical Sponges, Medical Cotton Tipped Applicators, Medical Cotton Balls, Medical Alcohol Prep Pads, Medical Alcohol Cotton Balls, Medical Iodine Cotton Balls, Medical Face Masks, Medical I.V. Adhesive Dressings, Human Chorionic Gonadotropin(HCG)Test Strips (colloidal gold), Luteinizing Hormone (LH)Test Strips (colloidal gold), Medical Fever Relief Patches, Gauze Bandages, Elastic Bandages, Medical Underpads, Medical Instant Cold Packs, Medical Exam Gloves, Liquid Dressings

Standards: EN ISO 13485:2016, MDD 93/42/EEC Annex V

Date: 2020-06-22~24
Report No.: 15081837 011

The result of on-site audit is positive. The auditors will recommend that TÜV Rheinland LGA Products GmbH Notified Body (CE0197) Certificate for a Quality Assurance System should be issued after the corrective actions proposed by the company are accepted.

Yours sincerely,
TÜV RHEINLAND (SHANGHAI) Co., Ltd.

Mr. SHENG Fuxiu
Leader Auditor, Greater China
Medical Services

TÜV Rheinland
(Shanghai) Co., Ltd.
TUV Building
No.177 Lane 777,
West Guangzhong Road,
Shanghai 200072,
P.R.China

Tel: +86-21-6108 1188
Fax: +86-21-6108 1359
+86-21-6108 1199

Hotline:
800 999 3568/400 883 1300
service-g@tuv.com.cn

www.tuv.com