

说明书

【产品名称】 创可贴

【产品型号】 C100 - C120

【产品规格】 72mm×22mm\70mm×18mm\22mm×18mm\41mm×18mm\60mm×70mm\70mm×90mm。

【产品性能、主要结构组成、成分】 创可贴由接触创面的敷垫、背贴和保护层（临用前去除）组成。不含药物。

【适用范围】 该产品主要用于真皮浅层及其以上的浅表性小创伤、擦伤等，为浅表创面、皮肤损伤提供愈合环境。※

本品过敏者禁止使用。

※本品为密封的产品，若发现包装纸破损或已打开请勿使用。

※使用本品前，应清洁伤口，以免汗水等影响产品粘性。

※伤口周围如毛发过长，使用本品前，请剔除毛发，防止粘贴不牢或揭除时因粘连毛发引起对皮肤的二次伤害。

※使用时先对伤口进行消毒处理，然后根据伤口的大小选择合适的创可贴。

※本品为一次性使用产品。

※本产品使用完后，请丢进垃圾箱。

【使用说明】

使用本品前，应先清洁和消毒伤口。沿箭头方向，剥开包装纸，将敷垫对准伤口部位，分先后把左右两面保护层除去，并固定位置，再按压周边部位以达到适度粘贴。为保持伤口卫生，请每天至少更换一次，并使用到伤口痊愈为止。

【产品维护和保养方法、特殊储存、运输条件、方法】 于常温干燥、无腐蚀性气体处密闭存放。禁止将产品从包装袋或盒中取出裸露在空气中存放。利用交通工具常温下运输，防止雨淋，禁止日光直接照射。

【生产日期、失效日期】 见包装

instructions

【product name】 Sterile Wound Plasters

【product model】 C100 – C120

【specifications of the products】 72mm×22mm \70mm×18mm \22mm×18mm \41mm×18mm\
60mm×70mm \70mm×90mm。

【product performance】 the main structure of the wound patch is made of contact pads, back patches and protective layers (removed prior to use) Contains no medication

【scope of application】 The products are mainly used in the superficial dermis and more superficial small wounds, abrasions, superficial wounds, skin injury healing environment.

※It is forbidden to use this product

※This product is a sealed product. If it is found damaged or opened, please do not use it

※Before using this product, clean the wound so as not to affect the stickiness of the product

※Around the wound such as hair is too long, use this product before, please remove hair, prevent sticking not firmly or remove because of adhesion, hair caused two damage to the skin

※When in use, please disinfect the wound first, then select the appropriate wound paste according to the size of the wound ※

This product is a one-time use of products

※After the product is finished, please throw it into the Dustbin.



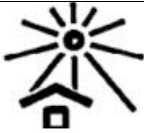


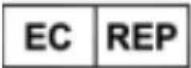




【instructions】

Before using this product, should first clean and disinfect the wound. Along the direction of the arrow and unwrapped it, the pad alignment of the wound site, have the right and left sides and remove the protective layer, fixed position, and then press the peripheral parts to achieve appropriate paste. To keep the wound health, please change at least once a day, and used to the wound until healed

【product maintenance and maintenance method, special storage, transportation conditions and methods】 Dry, non-corrosive gas sealed storage. Prohibit the product stored in the air exposed out of the packaging bag or box. The use of transportation under normal transportation, prevent rain, no direct sunlight

【date of production, date of expiry】 see packing

【医疗器械标签所使用的图形、符号、缩写等内容的解释】 [using the labels of medical devices graphics, symbols, abbreviations and other content explanation]

				
环氧乙烷灭菌 Sterile EO	不能 2 次使用 Cannot be used 2 times	避免阳光直射 keep out of the direct sun	保持干燥 Keep dry	请参考使用说明书 Please refer to the instruction manual
				
欧盟授权代表 European authorized representative	生产厂家 manufacturer	有效期 period of validity	生产日期 date of manufacture	批号 batch number

【注册人/生产企业名称】 青岛海诺生物工程有限公司 The registrant / manufacturer name: Qingdao Hainuo Biological Engineering Co. Ltd.

【注册人/生产企业住所】 青岛市莱西市姜山镇工业园广东路1号No.1 Guangdong Road, Jiangshan Industrial Zone, Laixi, Qingdao, China

【注册人/生产企业联系方式】 Contact person / production enterprise registration 0532—86463333/86464444

【售后服务单位】 after sale service 青岛海诺生物工程有限公司 Qingdao Hainuo Biological Engineering Co. Ltd.

【生产地址】 青岛市莱西市姜山镇工业园广东路1号No.1 Guangdong Road, Jiangshan Industrial Zone, Laixi, Qingdao, China

【CE 标志及审核机构代码】 CE logo and audit organization code



【欧盟代表】: 西班牙CMC医疗器械有限公司

【地址】: 霍拉西奥伦戈街18号, 马拉加, 西班牙

【说明书的编制日期】 2022 年 12 月 06 日

【说明书版本号】 第 2 版