

Instruction for Use	File No.:	HN-CE03-Annex-05
	Revision:	A/0
<i>CE Technical file of Fever Cooling Patch</i>	Effective:	2020.09.25



Instruction for Use

【Product Name】

Fever Cooling Patch

【Model/Size】

No.	Model	Size
1	C651	40mmx110mm
2	C652	50mmx12mm

【Intended use】

It is used for physical cooling of human body and cooling of specific parts of body surface.

【Material】

Hydrophilic molecules and macromolecule material, mint, ice sheet, water.

【Specification】


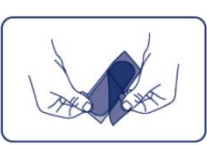


4cm×11cm

5cm×12cm

【Contraindications】

Do not use on damaged skin.

【Directions】

Step 1	Step 2	Step 3	Step 4
			

【Warnings】

- 1) Keep away from children.
- 2) Use only under adult supervision.
- 3) For external use only.
- 4) Stop use if skin allergy or irritation occurs.

Instruction for Use	File No.:	HN-CE03-Annex-05
	Revision:	A/0
<i>CE Technical file of Fever Cooling Patch</i>	Effective:	2020.09.25

- 5) Avoid contact with eyes and mouth.
- 6) If any adverse event occurs, it should be reported to the manufacturer and competent authority of the Member State where the user and/or patient is located.

【Package】

The inner packaging uses paper bags, the medium package is a white cardboard and 6 pcs per box.

【Expiry】

2 years.

【Production Date】

Refer to the package label.







【Storage conditions】

The product shall be stored in dark, cool place without direct sunlight.







【Sterilization】

Non-sterile

【Symbol Description】

	Caution, Indicates the need for the user to consult the instructions for use for important cautionary information, such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.
	CE Mark: conforms to essential requirements of the Medical Device Regulation 2017/745 (EU)
	UKCA Mark: conforms to essential requirements of the Medical Device Regulation Part II of the UK MDR 2002
	Date of manufacture, Indicates the date when the medical device was manufactured
	Manufacturer, Indicates the medical device manufacturer, as defined in Medical Device Regulation 2017/745 (EU)
	Use-by date, Indicates the date after which the medical device is not to be used

Instruction for Use	File No.:	HN-CE03-Annex-05
	Revision:	A/0
<i>CE Technical file of Fever Cooling Patch</i>	Effective:	2020.09.25

	Do not re-use, indicates a medical device that is intended for one use, or for use on a single patient during a single procedure
	Do not use if package is damaged, indicates a medical device that should not be used if the package has been damaged or opened
	Non-sterile
UDI-DI	UDI-DI code of product identification
LOT	Batch code, Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Keep away from sunlight, indicates a medical device that needs protection from light sources.
	Keep dry, indicates a medical device that needs to be protected from moisture.
MD	Medical Device mark
	Qingdao Hainuo Biological Engineering Co., Ltd. No.1 Guangdong Road, Jiangshan Industrial Park, Laixi, Qingdao, China.
EC REP	CMC Medical Devices & Drugs S.L C/Horacio Lengo N° 18 CP 29006, Málaga-Spain
UK REP	Kingsmead Service Limited 19 Mezzanine Floor 19-21 Crawford Street London England W1H 1PJ