

Instruction for Use

【Production Name】

Disposable Medical Face Mask

【Model】

C007\ C010\ C016\ C020

【Product feature, Main Structural Composition/Constituents】

It is not a form-shaped filtering device (i.e., not a respirator). It is made of a flexible, porous fabric or paper material and is typically secured using elastic head straps or ties; it may incorporate a forming nosepiece (metal wire) and/or transparent face/eye visor intended to protect the upper face/eyes from debris/fluid. This is a single-use device.

【Range of Application】

A flexible, loose-fitting mask designed to be placed over the mouth and/or nose of a healthcare worker to permit normal breathing while protecting the patient and wearer from the transfer of large particles (e.g., blood, body fluids, and airborne particulate materials) during medical procedures and patient examination.

【Contraindications】

- 1) Face masks are not intended for use as a chemical barrier. Do not use these masks for protection from hazardous chemicals. Facemasks should not be used near an open flame or any material that has the potential to ignite.
- 2) masks have a certain intended use only.
- 3) When the masks are not in use, avoid relative humidity > 80% and toxic gases.
- 4) This product is for one-time use only.
- 5) This product is for staff to use and destroy it according to environmental protection requirements after use.

【Storage Condition】

The masks shall maintain their properties when stored in a dry condition at temperature not higher than 50°C. Do not store in highly humid or damp Areas. Room temperature, do not expose to excessive heat, humidity, or direct sunlight.

【Production Address】

No.1 Guangdong Road, Jiangshan Industrial Park, Laixi, Qingdao, China, 266603

【Production Enterprise】

Qingdao Hainuo Biological Engineering Co., Ltd.

【Production Date / Batch Number, Service Life】

Refer to the Package.




【Product Validity】


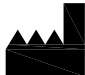










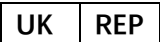
2 Years.

【Sterilization】

Non-sterile.

【Symbol Description】

	Caution, Indicates the need for the user to consult the instructions for use for important cautionary information, such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.
	CE Mark: conforms to essential requirements of the Medical Device Regulation 2017/745 (EU)
	UKCA Mark: conforms to essential requirements of the Medical Device Regulation Part II of the UK MDR 2002

	Date of manufacture, Indicates the date when the medical device was manufactured
	Manufacturer, Indicates the medical device manufacturer, as defined in Medical Device Regulation 2017/745 (EU)
	Use-by date, Indicates the date after which the medical device is not to be used
	Do not re-use, indicates a medical device that is intended for one use, or for use on a single patient during a single procedure
	Do not use if package is damaged, indicates a medical device that should not be used if the package has been damaged or opened
	Non-sterile
UDI-DI	UDI-DI code of product identification
	Batch code, Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Keep away from sunlight, indicates a medical device that needs protection from light sources.
	Keep dry, indicates a medical device that needs to be protected from moisture.
	Medical Device mark
	Qingdao Hainuo Biological Engineering Co., Ltd. No.1 Guangdong Road, Jiangshan Industrial Park, Laixi, Qingdao, China.
	CMC Medical Devices & Drugs S.L C/Horacio Lengo N° 18 CP 29006, Málaga-Spain
	Kingsmead Service Limited 19 Mezzanine Floor 19-21 Crawford Street London England W1H 1PJ